V4

CLEARMETAL MATRIX SYSTEM

INSTRUCTIONS FOR USE

 triodont®  
Innovative. Simple. Smart.

**Hello and thank you for purchasing the V4 ClearMetal Matrix System™. Before using it, please spare a moment to consider my reasons for developing this latest advance in matrix technology.**

The primary purpose of V4 is to allow sufficient light through so that initial curing occurs and the matrix can then be removed without risk of distorting the composite resin. A final cure after the matrix has been removed is always necessary and this will provide the complete cure. Getting past the initial cure stage for a deep restoration with bulk fill is a concern for most dentists. This is because removing a matrix that is in contact with uncured composite can be catastrophic, even requiring a complete second procedure, from prep to placement. Although this may not happen that often, the V4 ClearMetal Matrix System insures against it ever happening. We also need to remember that we have a metal matrix band that has all the other features that dentists want:

- Thin but strong
- Holds its shape but can be burnished
- A wonderfully, anatomically correct shape
- PTFE non-stick coated
- Has good contrast relative to the tooth so that it is easy to see

Mylar bands do not meet these requirements, which is why they are unpopular. Lastly, our ClearMetal Matrices are infinitely more transparent than other metal matrices! Thank you, and happy dentistry.

**Dr Simon McDonald** BSc MSc DDPH RCS  
Inventor, Triodent Sectional Matrix System

---

## SYSTEM ADVICE

---

### V4 RING

The tough and durable V4 Rings produce ideal tooth separation and outstanding grip, and the transparent tines present a cure-through option.

### CLEARMETAL MATRIX

The state-of-the-art ClearMetal Matrix has “micro-windows” that allow light to pass through, combining the advantage of transparency with the malleability of metal and high contrast of metal against the tooth. The ClearMetal Matrix is also non-stick for easy removal, and is highly anatomical to match the contours of the tooth.

### V4 WEDGE

The V4 Wedge is also transparent, and its segmental design cleverly facilitates excellent matrix adaptation and retention in all Class IIs, whether the cure-through option is used or not.

### PIN-TWEEZERS AND FORCEPS

Triodent Pin-Tweezers and Forceps are designed to hold and place Triodent rings, matrices and wedges with ease and security not matched by other tweezers, forceps and pliers. **We recommend that only Triodent Pin-Tweezers and Forceps are used with the V4 System.**

### PIN-TWEEZERS

Use Triodent Pin-Tweezers to avoid dropping matrices and wedges. Squeeze the tweezers at the back to open and near the tip to grip. When gripping the matrix, ensure that the gold side of the tweezers tips will be facing away from the tooth. Otherwise the pin can be difficult to disengage from the tab.

Check the gingival margin for gaps. If necessary, place a second wedge. You can do this with the ring and matrix in place, without starting over.

### INDICATIONS

The V4 ClearMetal Matrix System™ is indicated for all Class II restorations.

V4 features three main components: the V4 Ring™, the ClearMetal Matrix™, and the V4 Wedge™.

### CONTRAINDICATIONS

None known.

---

## SAFETY

---

For utmost safety and to gain maximum satisfaction and benefit from these products, carefully read these Instructions for Use in full and view the training animation on our website – [www.trident.com](http://www.trident.com).

**Note: Always refer to [www.trident.com/ifu](http://www.trident.com/ifu) for the latest Instructions for Use.**

### WARNINGS

The V4 ClearMetal Matrix System contains small, sharp parts. Use of rubber dam is mandatory to prevent the potential for inhalation or ingestion of such parts.

**Aspiration:** If inhalation of parts occurs, seek immediate medical attention.

**Ingestion:** If accidental swallowing occurs, encourage patient to seek medical assistance to locate swallowed part. If nausea or illness develops, seek medical attention immediately.

**Laceration:** The Trident matrices are sharp, and may cut unprotected skin. Handling matrices with gloved hands and/or pin tweezers or similar instrument is recommended. In the event a clean matrix causes a skin laceration, employ usual first aid measures. Discard and properly dispose of matrix.

**Handling:** Used matrices and accessories are contaminated. Follow proper exposure control plans. When handling or disposing, use precautions as when handling or disposing other contaminated sharps. In the event of an accidental exposure, follow recommended post-exposure procedures.

Care should be taken to have good control of any instrument to protect patients from injury.

### PRECAUTIONS

This product is intended to be used only as specifically outlined in the Instructions for Use. Any use of this product inconsistent with the Instructions for Use is at the discretion and sole responsibility of the practitioner.

**Small parts under tension:** Check that the ring is positioned correctly in the forceps before placement. Before withdrawing forceps, check that the ring is seated securely apically.

Tie dental floss to the V4 Ring for added security.

**Rings and instruments should be steam-autoclaved at 134°C/273°F for at least 3 minutes before use for sterility.**

**MANY CHEMICALS AND DETERGENTS ATTACK CLEAR PLASTIC. MINIMISE CONTACT OF THE V4 RING WITH ALL CHEMICAL AGENTS.**

**Do not use chemical sterilization techniques.**

Avoid contact with ketones.

Wear suitable protective eyewear and gloves.

Wedges and matrices are for single use only. Properly dispose of contaminated single-use items and sharps.

Do not use accessory tools other than those designed, supplied, and/or outlined for use with the V4 System. Injury may result. (See Step by Step Instructions).

### ADVERSE REACTIONS

Product misuse may cause failure resulting in respiratory, digestive, eye, skin, or mucosa injury. (See Contraindications, Warnings and Precautions).

Failure to employ accepted handling and disposal techniques may present biohazard. (See Warnings and Precautions).

---

## V4 SYSTEM INSTRUCTIONS

---

**Matrices and wedges are for single use only. Rings, Pin-Tweezers and Forceps are reusable and should be steam autoclaved at 134°C/273°F for at least 3 minutes prior to clinical use (refer Hygiene section).**

### ISOLATION

Always place rubber dam or other suitable isolation technique.

#### **Technique Tip: Placing wedge first vs matrix first**

Wedging before placing the matrix prevents bleeding during cavity prep and eliminates the possibility of a gingival tag protruding into the box or entrapment of rubber dam. If you prefer to place the matrix before the wedge, trap the matrix tab with your finger against the neighboring tooth while inserting the wedge. This prevents the matrix sliding out of the embrasure. Ensure there is no gap between the matrix and the tooth at the gingival margin after ring placement. If there is, engage one of the matrix wings with a suitable instrument and pull the matrix tight.

### WEDGE SELECTION AND PLACEMENT

Wedge size selection is important to obtain maximum adaptation and prevent wedge deformation. For optimum adaptation, choose the largest wedge size that will fit the interproximal space and slide it into place. If the wedge deforms, choose a smaller size.

#### **Technique Tip: Pre Load Pin-Tweezers**

To save time and avoid dropping matrices and wedges, ask your assistant to pre-load the appropriate sizes using Triodent Pin-Tweezers. Pin-Tweezers minimize the possibility of unintentional release, allowing for placement and removal of wedges and matrices in a safe and controlled manner. Squeeze the Pin-Tweezers at the back to open and near the tip to grip.

### CAVITY PREPARATION

Complete the cavity prep as usual.

### MATRIX SELECTION AND PLACEMENT

The 5.5mm is recommended for most applications. The 4.5mm is designed for use with adolescents and patients with poorly erupted posterior teeth. The 6.5mm is designed for larger cavities occlusogingivally.

After cavity prep, ensure that the interproximal contact is open. If there is no gap, we recommend using a diamond polishing tip to open it slightly. Now grip the tab of an appropriate sized ClearMetal Matrix with tweezers, ensuring that the gold side of the tweezer tips will be facing away from the tooth, bend the tab towards you to create a contra-angle, and place the matrix apically in the gingival crevice. To prevent the matrix moving when you release the tweezers, use a finger to trap one end against the buccal or lingual surface of the tooth.

#### **Technique Tip: Opening the contact**

To ensure more predictable results, open the contact area, using for example the V4 Ring or a diamond polishing strip. This makes it easier to position the matrix correctly without deformation, and to view the vertical margins during the procedure, and also to check these margins at follow-up appointments. Beveling the gingival margin will also make it easier to confirm by sight, inside the box, that you have a proper matrix seal.

### V4 RING SELECTION AND PLACEMENT

The V4 Molar Ring is indicated for most Class II cases but more specifically molars. The V4 Premolar Ring is designed for premolars and primary teeth as it has a stronger action to achieve optimum separation on narrow embrasures. The premolar ring can also be useful for teeth that are difficult to separate.

Select the appropriate sized V4 Ring. Using forceps (***we recommend that only Triodent's specially designed forceps should be used***), engage the slots on the inside of the ring, then spread the forceps to expand the ring just enough for the tines to place over the embrasure.

#### **Technique Tip: Ring orientation**

Often it is best to place the V4 Ring distally for better view and access for restoration and light curing, even with an MO.

Using a finger to steady the end of the forceps, slowly and carefully place the ring as low as possible with the tines straddling the wedge. Slowly release the forceps tension while using a finger to press the matrix tab down on to the occlusal surface of the adjacent tooth. This prevents movement of the matrix and possible spring-off of an incorrectly placed ring. When the ring is securely seated, remove the forceps.

When correctly placed, the tips of the tines should be close to the gingival margin, and the V-slot of the V4 Ring is touching the wedge such that the ring will not move any further in an apical direction.

#### **Technique Tip: Burnishing and contact tightness**

The matrix is already anatomically shaped so burnishing is not needed unless the clinician wants to customize the provided contour. All that is needed to ensure a tight contact point is for the matrix band to be touching the adjacent tooth prior to composite placement. This can be done carefully with a blunt instrument like a ball burnisher or the tips of Triodent Pin-Tweezers if they fit into the preparation.

#### **Technique Tip: Wedging from both sides**

The V4's V shaped tines allow you to wedge from both sides. This can be great for sealing the bottom axial corner of the box or to ensure that there is an optimal seal along the bottom edge of the box.

## **RESTORATION**

Make sure that the matrix is in full contact with the adjacent tooth and seals the entire margin of the box. Etch and bond and complete the restoration using your preferred composite. You may light-cure as normal, but with the “clear advantage” of the V4 System you also have the peace-of-mind option of light-curing through the ring, wedge and matrix. Cure through the buccal and lingual aspects, placing the face of the curing light flush against the side of the ring tine near the tip if possible, and then cure the occlusal aspect.

## **REMOVAL AND FINISHING**

Carefully remove the V4 components, taking special care with the matrix to prevent laceration of the gingiva. Give the restoration a final cure, and complete your finishing. If you are dealing with a very deep proximal cavity, an alternative strategy is to remove only the V4 Ring and then cure interproximally with the ClearMetal Matrix still in place. This allows the curing-light tip better access to the deepest part of the restoration and ensures that the composite is adequately cured prior to matrix removal.

**Note:** Over-tight contacts will resolve themselves in a day or two through minor (<30µm) tooth movements similar to orthodontic movements. No awkward disking or stripping of contacts is necessary.

---

## **CLINICAL TIPS**

---

### **RESTORING MODs**

The recommended method is as follows:

- 1 Prepare M and D cavities
- 2 Place matrices and wedges in M and D, and etch and bond
- 3 Place V4 Ring on M; build and cure M contact point
- 4 Move V4 Ring to D; build and cure D contact point
- 5 Complete occlusal surface and cure with C-Factor issues in mind

### **MOD ring orientation**

The V4 Ring may be placed facing mesially or distally. In an MOD, the rings may be stacked, with both rings facing in the same direction, or placed in opposite directions to give a more open operative field.

### **Restoring back to back MO and DO**

The easiest method for back-to-back Class II restorations is to complete one and then do the other. Doing both simultaneously can be faster but requires more operator skill. A V4 Premolar Ring is recommended as more separation is necessary to overcome two thicknesses of matrix.

### **TOOTH SEPARATION WITH V4**

Unlike a traditional wedge, the V4 Wedge will not separate the teeth to any significant degree. Its function is primarily to secure, adapt and seal the

matrix below the gingival margin. The V4 Ring has been designed to apply the ideal force required to separate the teeth sufficiently to easily compensate for the thickness of the matrix and achieve a predictable, tight contact area.

---

## TRIODENT V4 RING HYGIENE

---

### CROSS CONTAMINATION

Do not reuse used matrices or wedges. Properly dispose of the used and/or contaminated matrices and wedges in accordance with local regulations. During incineration, wedges may produce irritating and highly toxic gases.

### REMOVING CONTAMINANTS

It is strongly recommended that rings are reprocessed as soon as is reasonably practical following use in order to facilitate cleaning and minimise the likelihood of restoration material adhesion and build-up on the tines.

### PRE-STERILIZATION

Rinse all excess soil from the ring.  
Gently scrub the ring using a soft nylon brush in warm water.  
If dental restorative materials have adhered to the surface of the tines, a flat blunt plastic instrument can be used to wedge and gently lift them off.  
Afterward, rinse rings in warm water.

### ULTRASONIC CLEANING

**Many chemicals, including detergents, attack clear plastic. If you wish to ultrasonically clean the ring we recommend putting the ring in a beaker with water and placing that in the ultrasonic cleaner to prevent contact with chemicals.**

### AUTOMATED WASHER DISINFECTOR

#### THE USE OF AN AUTOMATIC WASHER DISINFECTOR IS NOT RECOMMENDED FOR THE V4 RINGS.

Triodent placement forceps and pin tweezers can be reprocessed in a washer-disinfector.

1. Use only properly maintained, calibrated, and approved washer-disinfector according to ISO 15883-1\*.
2. Place devices in the washer-disinfector allowing water and detergent to enter and drain off from device openings and follow washer-disinfector's Instructions for Use.
3. Run washer-disinfector program with A0 value  $\geq 3000$  (e.g. 5 min at  $\geq 90^{\circ}\text{C}$ ) using appropriate detergents\*.

Devices can be reprocessed by washer-disinfector for up to 500 times.

\*For example Miele Washer-Disinfector G7835 CD using program DESIN vario TD with neodisher® MediClean forte (alkaline detergent) and neodisher®

Z (acid neutralization and cleaning detergent); both detergents from Dr. Weigert, Hamburg, Germany (not products or registered trademarks of Dentsply Sirona).

### STERILIZATION

Steam-autoclave V4 Rings at  $134^{\circ}\text{C}/273^{\circ}\text{F}$  for at least 3 minutes. To prevent scratching of the rings, keep them separate from other instruments in the autoclave. **Do not use chemical sterilization techniques. Chemical sterilization will void warranty.**

**Note:** Use of other sterilization procedures/methods is at the discretion and sole responsibility of the practitioner.

### POST STERILIZATION

Allow forceps, pin tweezers and retaining rings to dry prior to storage. Forceps and pin tweezers require no additional maintenance and should not be oiled or greased.

### STORAGE

Store in a clean, dry environment away from moisture or excessive humidity. Keep out of sunlight. Keep away from flame or source of ignition.

---

## TRIODENT V4 RING MAINTENANCE

---

Before use, visually inspect the tines for damage, such as cracking or pitting.

### RING LONGEVITY

To help ensure you get the longest possible use from your rings:

- Avoid over-stretching. Open the ring only as far as needed to place it over the tooth.
- Do not leave the ring open in the forceps for a long period of time.
- **Do not use chemical sterilization techniques.**

**MANY CHEMICALS AND DETERGENTS ATTACK CLEAR PLASTIC. MINIMISE CONTACT OF THE V4 RING WITH ALL CHEMICAL AGENTS.**

---

## CORRESPONDENCE

---

To help us assist you the following numbers should be quoted in all correspondence:

- Reorder Number
- Lot number on the package
- Copy of proof of purchase

### MORE INFORMATION

For full product information and latest Instructions For Use go to:  
[www.trident.com/ifu](http://www.trident.com/ifu)

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Dentsply DeTrey GmbH