
Innovative. Simple. Smart.

V3 INSTRUCTIONS FOR USE

INSTRUCTIONAL GUIDE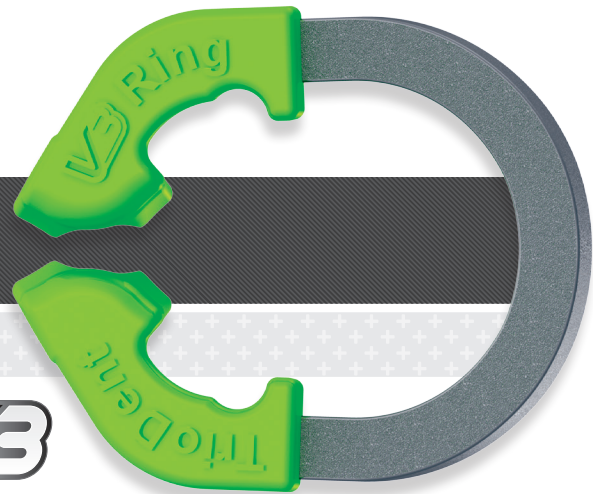

THANK YOU FOR PURCHASING THE MULTI AWARD-WINNING V3 SECTIONAL MATRIX SYSTEM

The V3 Ring produces high-quality, anatomically accurate restorations. The nickel-titanium ring has exceptional strength for stability and exerts optimal separation force on the teeth to produce consistently tight contacts. The V-shaped tines ensure there is no competition with the wedge in the embrasure, and they grip both teeth equally, preventing the ring from collapsing into wide cavities.

Please carefully read all of the following information to ensure that you and your patients gain the maximum benefit and satisfaction from all of the simple and smart innovations we have designed within the V3 System.

SYSTEM ADVICE

V3 RING

The tough and durable V3 Ring produces ideal tooth separation and outstanding grip.

V3 TAB MATRIX

The V3 Tab-Matrix is the most anatomically accurate matrix on the market. Its range of features are there to produce the most natural restorations possible, while at the same time making the job of the dentist much easier.

WAVE WEDGE

Used with the V3 Ring, the Wave-Wedge has unsurpassed sealing capability at the gingival margin. Its smart features, including a central concavity – the ‘wave’ – ensure it stands head and shoulders above any other non Triodent wedge available.

WEDGE GUARD

WedgeGuards minimise the risk of damaging the adjacent tooth and the papilla so you can do your preps better and faster at the same time. After cavity prep, the guard is detached while the wedge stays in place, ready for the insertion of a matrix beside it. Leaving the wedge in place prevents reactive hyperemia, ensuring a cleaner working area.

PIN-TWEEZERS AND FORCEPS

Triodent Pin-Tweezers and Forceps are designed to hold and place Triodent rings, matrices and wedges with ease and security not matched by other tweezers, forceps and pliers.

We recommend that only Triodent Pin-Tweezers and Forceps are used with the V3 System.

PIN-TWEEZERS

Use Triodent Pin-Tweezers to avoid dropping matrices and wedges. Squeeze the tweezers at the back to open and near the tip to grip. When gripping the matrix, ensure that the gold side of the tweezers tips will be facing away from the tooth. Otherwise the pin can be difficult to disengage from the tab.

Check the gingival margin for gaps. If necessary, place a second wedge. You can do this with the ring and matrix in place, without starting over.

INDICATIONS

The V3 Sectional Matrix System™ is indicated for all Class II restorations.

V3 features three main components: the V3 Ring™, the V3 Tab-Matrix™, and the Wave-Wedge™, and can also be used with WedgeGuard™.

CONTRAINDICATIONS

None known.

SAFETY

For utmost safety and to gain maximum satisfaction and benefit from these products, carefully read these Instructions for Use in full and view the training animation on our website – www.trident.com.

Note: Always refer to www.trident.com/ifu for the latest Instructions for Use.

WARNINGS

The V3 Sectional Matrix System contains small, sharp parts. Use of rubber dam is mandatory to prevent the potential for inhalation or ingestion of such parts.

Aspiration: If inhalation of parts occurs, seek immediate medical attention.

Ingestion: If accidental swallowing occurs, encourage patient to seek medical assistance to locate swallowed part. If nausea or illness develops, seek medical attention immediately.

Laceration: The Trident matrices are sharp, and may cut unprotected skin. Handling matrices with gloved hands and/or pin tweezers or similar instrument is recommended. In the event a clean matrix causes a skin laceration, employ usual first aid measures. Discard and properly dispose of matrix.

Handling: Used matrices and accessories are contaminated. Follow proper exposure control plans. When handling or disposing, use precautions as when handling or disposing other contaminated sharps. In the event of an accidental exposure, follow recommended post-exposure procedures.

Care should be taken to have good control of any instrument to protect patients from injury.

PRECAUTIONS

This product is intended to be used only as specifically outlined in the Instructions for Use. Any use of this product inconsistent with the Instructions for Use is at the discretion and sole responsibility of the practitioner.

Small parts under tension: Check that the ring is positioned correctly in the forceps before placement. Before withdrawing forceps, check that the ring is seated securely apically.

Tie dental floss to the V3 Ring for added security.

Rings and instruments should be steam-autoclaved at 134°C/273°F for at least 3 minutes before use for sterility.

Do not use chemical sterilization techniques.

Ensure that phosphoric acid etchants are removed from the tines prior to steam sterilization.

Wear suitable protective eyewear and gloves.

Wedges and matrices are for single use only. Properly dispose of contaminated single-use items and sharps.

Do not use accessory tools other than those designed, supplied, and/or outlined for use with the V3 System. Injury may result. (See Step by Step Instructions).

ADVERSE REACTIONS

Product misuse may cause failure resulting in respiratory, digestive, eye, skin, or mucosa injury. (See Contraindications, Warnings and Precautions).

Failure to employ accepted handling and disposal techniques may present biohazard. (See Warnings and Precautions).

V3 SYSTEM INSTRUCTIONS

Matrices and wedges are for single use only. Rings, Pin-Tweezers and Forceps are reusable and should be steam autoclaved at 134°C/273°F for at least 3 minutes prior to clinical use (refer Hygiene section).

ISOLATION

Always place rubber dam or other suitable isolation technique.

Technique Tip: Placing wedge first vs matrix first

Wedging before placing the matrix prevents bleeding during cavity prep and eliminates the possibility of a gingival tag protruding into the box or entrapment of rubber dam. Even better, a WedgeGuard will protect the papilla/rubber dam and the adjacent tooth as well. If you prefer to place the matrix before the wedge, trap the matrix tab with your finger against the neighboring tooth while inserting the wedge. This prevents the matrix sliding out of the embrasure. Ensure there is no gap between the matrix and the tooth at the gingival margin after ring placement. If there is, engage one of the matrix wings with a suitable instrument and pull the matrix tight.

WEDGE SELECTION AND PLACEMENT

Wedge size selection is important to obtain maximum adaptation and prevent wedge deformation. For optimum adaptation, choose the largest wedge size

that will fit the interproximal space and slide it into place. If the wedge deforms, choose a smaller size.

Technique Tip: Pre Load Pin-Tweezers

To save time and avoid dropping matrices and wedges, ask your assistant to pre-load the appropriate sizes using Triodent Pin-Tweezers. Pin-Tweezers minimize the possibility of unintentional release, allowing for placement and removal of wedges and matrices in a safe and controlled manner. Squeeze the Pin-Tweezers at the back to open and near the tip to grip.

CAVITY PREPARATION

Complete the cavity prep as usual.

REMOVE GUARD (if applicable)

If using a WedgeGuard, detach the guard after prep, leaving the wedge behind. A mirror can be used to hold the wedge in place while the guard is removed.

MATRIX SELECTION AND PLACEMENT

The 5.5mm is recommended for most applications. The 4.5mm and 3.5mm are designed for use with adolescents and patients with poorly erupted posterior teeth. The 6.5mm is designed for larger cavities occlusogingivally.

After cavity prep, ensure that the interproximal contact is open. If there is no gap, we recommend using a diamond polishing tip to open it slightly. Now grip the tab of an appropriate sized matrix with

tweezers, ensuring that the gold side of the tweezer tips will be facing away from the tooth, bend the tab towards you to create a contra-angle, and place the matrix apically in the gingival crevice. To prevent the matrix moving when you release the tweezers, use a finger to trap one end against the buccal or lingual surface of the tooth.

Technique Tip: Opening the contact

To ensure more predictable results, open the contact area, using for example the V3 ring or a diamond polishing strip. This makes it easier to position the matrix correctly without deformation, and to view the vertical margins during the procedure, and also to check these margins at follow-up appointments. Bevelling the gingival margin will also make it easier to confirm by sight, inside the box, that you have a proper matrix seal.

V3 RING SELECTION AND PLACEMENT

The Universal V3 Ring (green) is indicated for most Class II cases but more specifically molars. The Narrow V3 Ring (yellow) is designed for premolars and primary teeth as it has a stronger action to achieve optimum separation on narrow embrasures. The yellow ring can also be useful for teeth that are difficult to separate.

Select the appropriate sized V3 Ring. Using forceps (***we recommend that only Triodent's specially designed forceps should be used***), engage the slots on the inside of the ring, then spread the

forceps to expand the ring just enough for the tines to place over the embrasure.

Technique Tip: Ring orientation

Often it is best to place the V3 Ring distally for better view and access for restoration, even with an MO.

Using a finger to steady the forceps, slowly and carefully place the ring as low as possible with the tines straddling the wedge. Slowly release the forceps tension while using a finger to press the matrix tab down on to the occlusal surface of the adjacent tooth. This prevents movement of the matrix and possible spring-off of an incorrectly placed ring. When the ring is securely seated, remove the forceps.

When correctly placed, the tips of the tines should be close to the gingival margin, and the V-slot of the V3 Ring is touching the wedge such that the ring will not move any further in an apical direction.

Technique Tip: Burnishing and contact tightness

The matrix is already anatomically shaped so burnishing is not needed unless the clinician wants to customize the provided contour. All that is needed to ensure a tight contact point is for the matrix band to be touching the adjacent tooth prior to composite placement. This can be done carefully with a blunt instrument like a ball burnisher or the tips of Triodent Pin-Tweezers if they fit into the preparation.

Technique Tip: Wedging from both sides

The V3's V shaped tines allow you to wedge from both sides. This can be great for sealing the bottom axial corner of the box or to ensure that there is an optimal seal along the bottom edge of the box.

RESTORATION

Make sure that the matrix is in full contact with the adjacent tooth and seals the entire margin of the box. Etch and bond and complete the restoration using your preferred composite, filling and curing incrementally if required.

REMOVAL AND FINISHING

Carefully remove the V3 components, taking special care with the matrix to prevent laceration of the gingiva. To ensure complete curing of the composite, carefully fold the wings of the matrix back and cure from the buccal and lingual sides.

Break the bond between the matrix and the composite by pushing a thin, metal blade instrument through the embrasure below the contact point.

Grip one of the matrix wing holes with Pin-Tweezers and wriggle and rotate the matrix gently around the contact point. When the matrix moves freely it can be removed.

Give the restoration a final light cure, and complete your finishing.

Note: Over-tight contacts will resolve themselves in a day or two through minor (<30µm) tooth movements, similar to orthodontic movements. No awkward disking or stripping of contacts is necessary.

CLINICAL TIPS

RESTORING MODs

The recommended method is as follows:

- 1 Prepare M and D cavities
- 2 Place matrices and wedges in M and D, and etch and bond
- 3 Place V3 Ring on M; build and cure M contact point
- 4 Move V3 Ring to D; build and cure D contact point
- 5 Complete occlusal surface and cure with C-Factor issues in mind

MOD ring orientation

The V3 Ring may be placed facing mesially or distally. In an MOD, the rings may be stacked, with both rings facing in the same direction, or placed in opposite directions to give a more open operative field.

Restoring back to back MO and DO

The easiest method for back-to-back Class II restorations is to complete one and then do the other. Doing both simultaneously can be faster but requires more operator skill. A Narrow V3 Ring (yellow) is recommended as more separation is necessary to overcome two thicknesses of matrix.

TOOTH SEPARATION WITH V3

Unlike a traditional wedge, the Wave Wedge will not separate the teeth to any significant degree. Its function is primarily to secure, adapt and seal the matrix below the gingival margin. The V3 Ring has been designed to apply the ideal force required to separate the teeth sufficiently to easily compensate for the thickness of the matrix and achieve a predictable, tight contact area.

TRIODENT V3 RING HYGIENE

CROSS CONTAMINATION

Do not reuse used matrices or wedges. Properly dispose of the used and/or contaminated matrices and wedges in accordance with local regulations. During incineration, wedges may produce irritating and highly toxic gases.

PREVENTING CONTAMINATION

A light smear of petroleum jelly on the V3 Ring tines before use will help prevent bonding agents and composites sticking.

REMOVING CONTAMINANTS

It is strongly recommended that rings are reprocessed as soon as is reasonably practical following use in order to facilitate cleaning and minimise the likelihood of restoration material adhesion and build-up on the tines.

PRE-STERILIZATION

Rinse all excess soil from the ring. Gently scrub the ring using a soft nylon brush in warm water. Afterward, rinse rings in warm water.

ULTRASONIC CLEANING

Most ultrasonic cleaning solutions will not affect the V3 Ring, although chemicals in some cleaners may cause tine discoloration. The discoloration is only cosmetic, but if that is a concern put the rings in a beaker with a water/detergent solution and place that in the ultrasonic cleaner to prevent contact with chemicals.

AUTOMATED WASHER DISINFECTOR

Triodent placement forceps, pin tweezers and retaining rings can also be reprocessed in a washer-disinfector.

1. Use only properly maintained, calibrated, and approved washer-disinfector according to ISO 15883-1*.
2. Place devices in the washer-disinfector allowing water and detergent to enter and drain off from device openings and follow washer-disinfector's Instructions for Use.
3. Run washer-disinfector program with A0 value ≥ 3000 (e.g. 5 min at $\geq 90^{\circ}\text{C}$) using appropriate detergents*. Devices can be reprocessed by washer-disinfector for up to 500 times.

*For example Miele Washer-Disinfector G7835 CD using program DESIN vario TD with neodisher® MediClean forte (alkaline detergent) and neodisher® Z (acid neutralization and cleaning detergent); both detergents from Dr. Weigert, Hamburg, Germany (not products or registered trademarks of Dentsply Sirona).

STERILIZATION

Steam-autoclave V3 Rings at $134^{\circ}\text{C}/273^{\circ}\text{F}$ for at least 3 minutes. Avoid leaving the rings in an autoclave overnight or for long periods, as this may cause staining from the autoclave itself.

Do not use chemical sterilization techniques.

Chemical sterilization will void warranty.

Note: Use of other sterilization procedures/methods is at the discretion and sole responsibility of the practitioner.

POST STERILIZATION

Allow forceps, pin tweezers and retaining rings to dry prior to storage. Forceps and pin tweezers require no additional maintenance and should not be oiled or greased.

STORAGE

Store in a clean, dry environment away from moisture or excessive humidity. Keep out of sunlight. Keep away from flame or source of ignition.

TRIODENT V3 RING MAINTENANCE

Before use, visually inspect the tines for damage, such as cracking or pitting.

RING LONGEVITY

To help ensure you get the longest possible use from your rings:

- Avoid over-stretching. Open the ring only as far as needed to place it over the tooth.
- Do not leave the ring open in the forceps for a long period of time.
- **Do not use chemical sterilization techniques.**

CORRESPONDENCE

To help us assist you the following numbers should be quoted in all correspondence:

- Reorder Number
- Lot number on the package
- Copy of proof of purchase

MORE INFORMATION

For full product information and latest Instructions For Use go to:

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Dentsply DeTrey GmbH